


The Transfer Handle® For Spring Style Hospital Beds

Models 2025H & 4025H

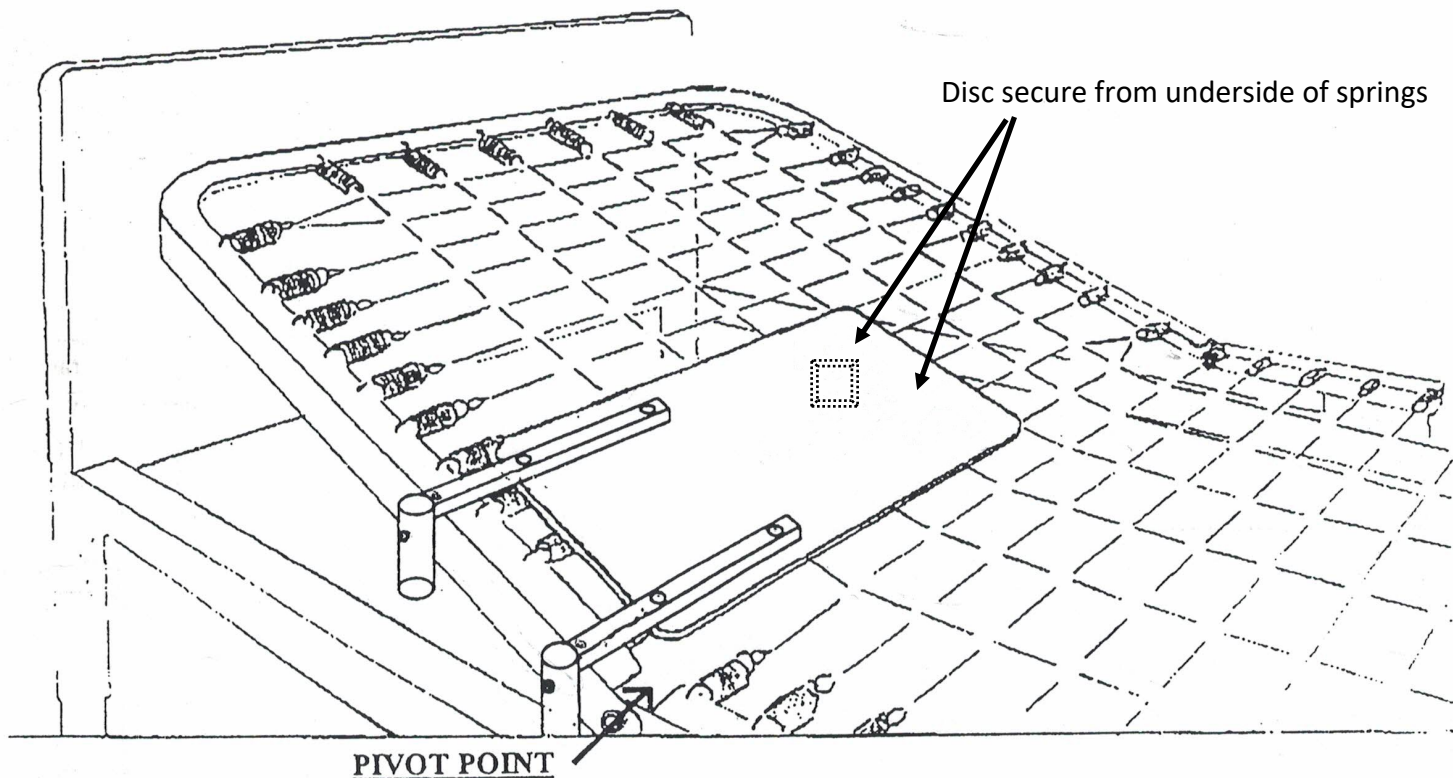

DIRECTIONS

1. PLACE HANDLE SUPPORT BOARD IN POSITION JUST ABOVE THE PIVOT POINT OF THE HEAD OF THE BED.

2. SECURE THE 2 DISCS W/KNOBS FROM THE UNDERSIDE OF THE BED SPRINGS.

3. INSTALL HANDLE (NOT SHOWN) INTO RECEIVING TUBES MAKING SURE SNAP BUTTONS LOCK IN PLACE.

