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item # 13004

**drive**  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

## installation instructions

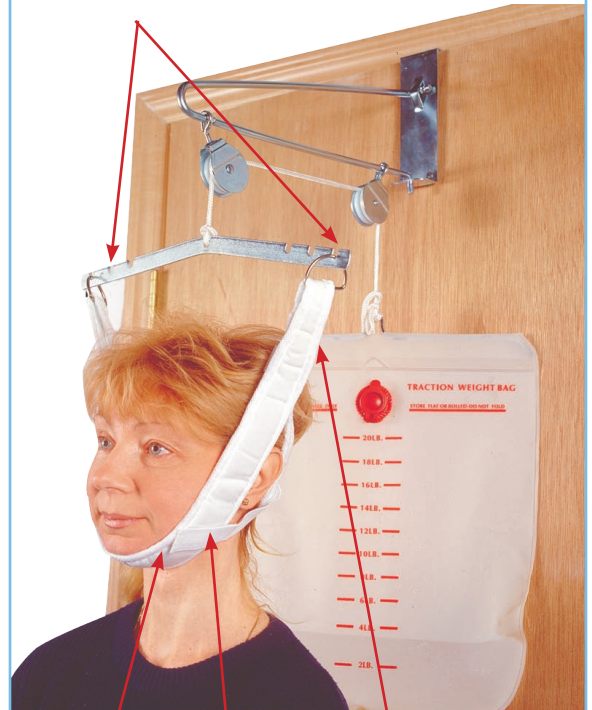
Weight Capacity: 20 lbs

1. Pick up head halter by two metal “D” rings, holding both with one hand, to make head halter hang as shown.
2. Place head through the middle with chin resting in chin cup and straps hanging from back of head “straps on outside”.
3. Fasten Velcro-type straps onto Chin Piece.
4. Adjust for correct size by moving overhead strap up or down at “D” rings.

**NOTE:** Adjust overhead strap as necessary.

Item # 13004  
Shown

“D” Rings

Chin Cup  
Front of Halter

Velcro-type  
Straps

Back Part of  
Halter with  
Straps on Outside

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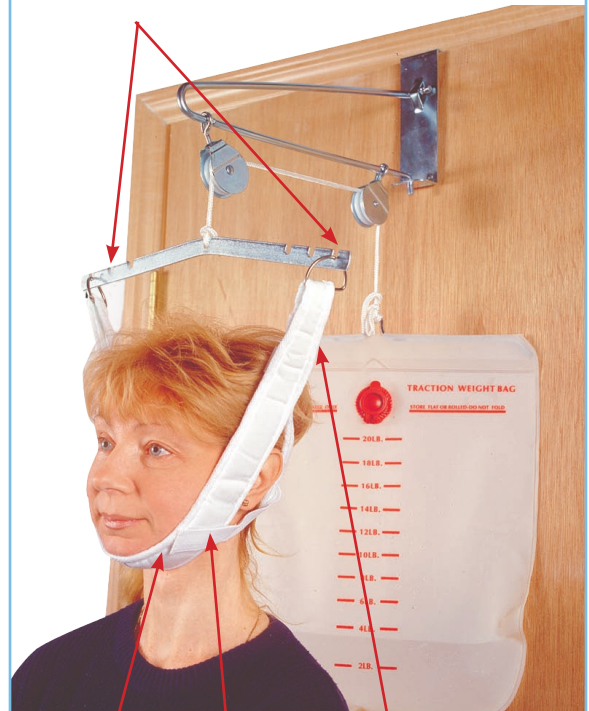
Bandes  
attaches Velcro-type

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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© 2006 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2006 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.