

The Transfer Handle® For Pan Style Hospital Beds

Models 6025H & 8025H


Installation Instructions:

1. When clamping to bed make sure Z Clip (A) captures the downward leg of the bed pan surface.

Note: Model 6025H uses a J Hook on the non handle side to secure the unit to the bed.



Figure 1

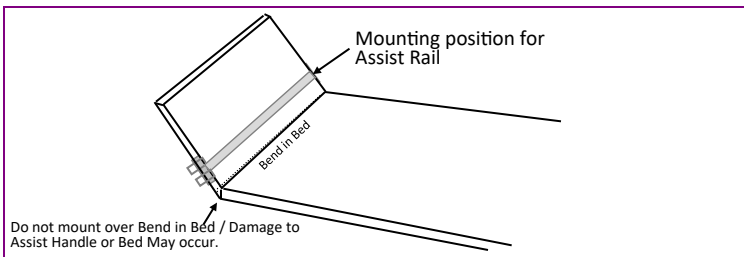

2. Position unit just above the hinge (C) on the upper portion of the bed. This is the hinge that allows the bed to ratchet up and down.

Important: Do not overlap this hinge with the mounting base. This will prevent the bed from ratcheting up and down.

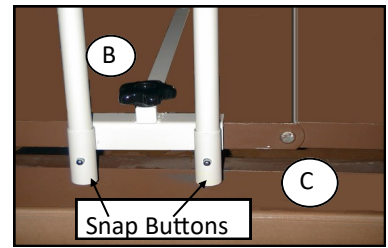


Figure 2

3. When attaching second side, insert center tube (D) into both of the square tubes.

4. When both sides are positioned properly tighten five star knobs (B). These knobs should be tightened securely so the unit has no slippage. Re-check tightness periodically.

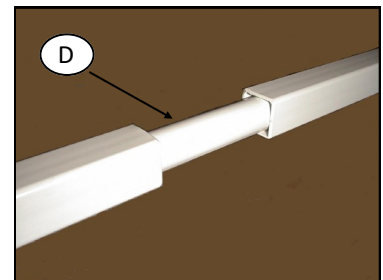


Figure 3

5. Insert handle into receiving tubes allowing snap buttons to lock into position.

Attention: CAUTION

Entrapment Potential: once installed be sure all dimensions are within your facilities guidelines. If the handle does not meet the guidelines, do not use.

Caution: If this patient assist handle does not appear to be attached securely, remove immediately from the bed and

