

Economy Bed Handle Bed Assist Rail

Model - 401



Image 1

Handle shown assembled and installed on bed. Position Bed Handle as to not hinder the patients ability to get in and out of bed if you wish them to be independent when transferring.

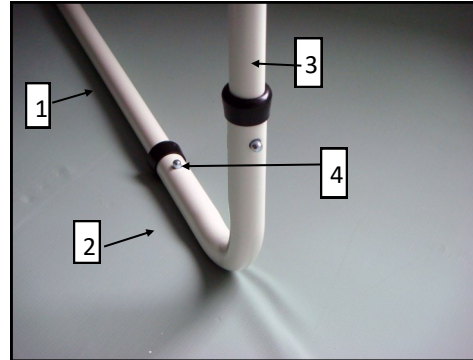


Image 2

Product Assembly. Number 1, base rail that slides under the mattress with securing strap. Number 2, the elbow connector which is preinstalled on the handle. Rotate the elbows until snap button locks in place. Number 3, Bed Rail handle. Number 4, shows the locking snap buttons locked in place. Once assembly is completed, slide base unit under mattress positioning the handle .



Image 3

Extend the securing strap that is attached to the base rail to the far side of the bed and secure to bed frame. Cinch the strap tight making sure the Bed Handle is tight to the side of the bed. Strap must be kept tight.

Warning: This product is not recommended for individuals with cognitive problems (difficulty in understanding their surroundings). If you are taking medications that affects your balance or your ability to sit or stand you should consult with a physician before using this product. If the product is not properly used, entrapment, serious injury or death could result. This product is not intended to carry the full weight of an individual. The product is only intended to assist in bed mobility and rising to a sitting position by providing a handle. These instructions must be carefully followed to insure that the product is properly assembled, installed and used for the stated purpose. If you have and questions or concerns please contact MTS at 1-800-854-4687.

Not for use on hospital style beds.

1003 Rev 01-17